SolaTone™ Electrolarynx

(GB) English

(E) Español

(P) Português

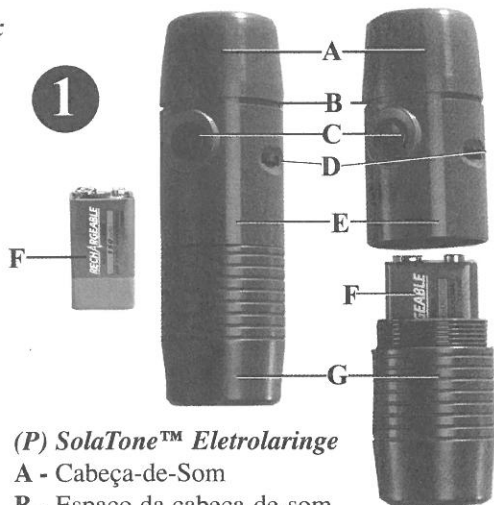
Griffin Laboratories (800)330-5969 (USA/Canada)
www.griffinlab.com

Users Guide 3-2001
Made in the USA - Patent Pending

7B) *SolaTone™ Electrolarynx*

- Sound-Head
- Sound-Head/Body Gap
- On/Off Activation Button
- Volume Knob
- Body
- Battery
- Battery Cap

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(GB) *Batteries*

1. Use 9 volt style rechargeable or disposable batteries.
2. Install battery(F) as shown. Match Positive(+) and Negative(-) Battery Contacts(H).
3. Use until volume fades and charge Rechargeable Batteries 16 -18 hours.
4. Do not recharge disposable batteries.

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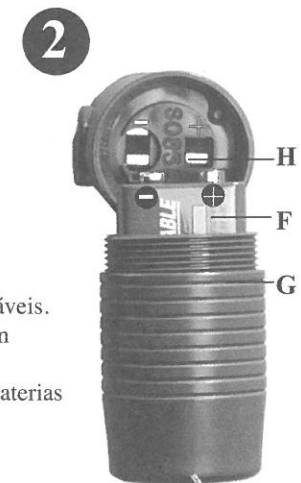
2

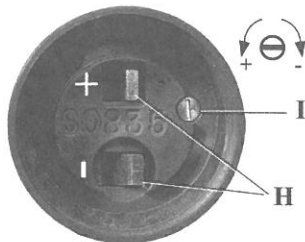
7B) Adjusting Tone

Tone Adjustment(I) is located next to the battery contacts(H).

Turn adjustment with screwdriver. Counterclockwise for higher tone. Clockwise to lower tone.

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3

(GB) Volume Level and Sound-Head/Body Gap

1. Set the volume level by turning Volume Knob(D).

2. Next, turn the Sound-Head(A) to increase or decrease the Sound-Head/Body Gap(B) for the best sound.

3. Sound-Head/Body Gap usually about 1 mm.

Sound-Head/Body Gap smaller at low volumes.

4. Also adjust when batteries are fresh or old.

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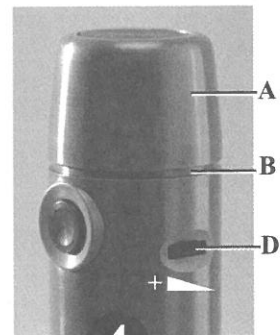
4

(GB) Electrolarynx Use On The Neck

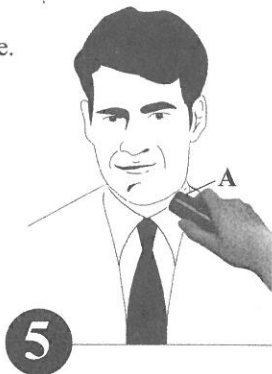
1. Place the Sound-Head(A) against your neck and activate.
2. Speak as sound is carried into your mouth.
3. Ensure a solid airtight contact on your neck and try different positions to get the best sound.
4. Use the oral adapter in cases of a tender neck or excess buzzing.

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(GB) Optional Oral Adapter & Straw

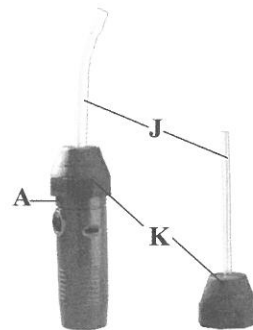
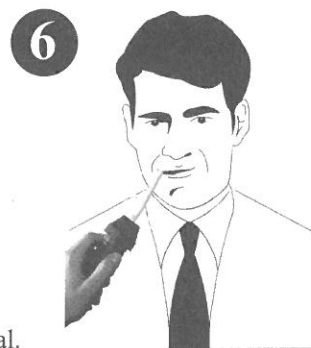
1. Insert Oral Straw(J) into top of Oral Cap(K).
2. Place Oral Cap over Sound-Head(A).
3. Place Oral Straw in mouth.
4. Activate Electrolarynx. Speak around Oral Straw.

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B) General Care and Troubleshooting

Cleaning - Wipe with a damp cloth. Do not use Oils, Electrical Contact Cleaners or Chemicals as they may cause damage.

Water Damage - Remove the caps from the body and air dry for 2 days before testing.

Low Volume/Muffled Sound - Adjust Sound-Head/Body Gap(B). Try a fresh battery.

Excessive Buzzing - Check complete seal on neck. Try different positions. Adjust the tone, volume level, and Sound-Head/Body Gap(B). Try the Oral Adapter.

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SolaTone™ Electrolarynx Warranty Registration Card

Name _____ Telephone _____

Address _____ City _____

State _____ Zip Code _____

Date Purchased _____ Dealer _____

Purchase Price _____ Speech Aid Serial # _____

Where you able to adjust your tone setting? _____ Yes _____ No

Do you understand the 16 - 18 hour battery charging procedure? _____ Yes _____ No

Have you adjusted the Sound-Head Gap for the clearest sound? _____ Yes _____ No

Would you like a Griffin Laboratories representative to call you? _____ Yes _____ No

Griffin Laboratories

27636 Ynez Rd., #L7199, Temecula, CA 92591, 800-330-5969

www.griffinlab.com

Complete and return warranty registration card within 45 days of original purchase date.

Place
Stamp
Here

Postage Required

Griffin Laboratories
Warranty Registration and Service
27636 Ynez Rd., #L7199
Temecula, CA 92591
USA

Limited Warranty

The *SolaTone*[™] Electrolarynx is guaranteed to be free of defects in parts and workmanship for a period of one(1) year from the date of purchase. Accessories included by Griffin Laboratories with the *SolaTone*[™] Electrolarynx are guaranteed against manufacturer defects for a period of sixty(60) days. To validate warranty you must complete and return the warranty card within forty five(45) days of the original purchase date from an authorized distributor.

This warranty is not transferable and does not cover damage caused by abuse, tampering, water damage, impact, soiling or neglect. This warranty is not valid outside the United States or Canada. Items qualifying for warranty coverage will be repaired or replaced at the discretion of Griffin Laboratories. All other warranties, either explicit or implied, are hereby excluded. Griffin Laboratories shall not be held liable for damages, either direct, indirect, or consequential, resulting from any defect in the manufacture or operation of the *SolaTone*[™] Electrolarynx.

Any electrolarynx requiring warranty service must be delivered with postage paid in a padded and insured package to Griffin Laboratories. Include a description of the problem, date of purchase, owners name, return address and telephone number. Griffin Laboratories will provide return shipping only within the United States and Canada. Contact Griffin Laboratories directly for shipping instructions and return authorization.

Griffin Laboratories 800-330-5969(USA/CANADA)
www.griffinlab.com