

## SoniVox® Plus™ Amplifier Instructions

### Instructions

Please read this entire instruction sheet prior to operating the SoniVox® Plus™ Amplifier system. We recommend you keep these instructions for future reference.

### Kit Components

SoniVox® Plus™ Amplifier, 5v power supply, Headset microphone, Neck Strap

### Extra Accessories (sold separately)

Auxiliary cable, Auxiliary extension cable, Clip on Microphone

### Neck Strap Installation

- 1) Remove the included neck strap from the kit and slide each end of the strap through the sliver side slots on the amplifier and back up through the adjustment pieces on the sides of the strap.
- 2) Adjust the strap to desired height and comfort.

### Charging Instructions

#### NOTE:

*The rechargeable Battery in this kit requires a full 10 hour charge before use.*

- 1) Plug the power supply into a wall outlet. The LED on the power supply will illuminate red while it is plugged in and is receiving power.
- 2) Locate the micro USB port labeled **CHARGE**; plug the included power supply into this port. The LED will illuminate red while the battery is charging.
- 3) The LED will illuminate green when the battery is fully charged.

### Microphone Installation

Locate the microphone jack on the amplifier (labeled **MIC**). Choose one of the microphones included in the kit and insert the plug into the MIC jack. **NOTE:** For best results, we recommend the headset microphone, especially for use by Laryngectomees. This microphone should be placed near the lips.

## SoniVox® Plus™ Amplifier Instructions

### Operating Instructions

- 1) Plug-in an included microphone and place it accordingly.
- 2) Turn the amplifier on by Pressing the **Power(Push)** knob down until the blue LED illuminates.
- 3) Adjust to desired volume by rotating the **Power(Push)** knob.

### Troubleshooting

Trouble	Solution
No Sound	<ul style="list-style-type: none"> <li>• Turn amplifier ON.</li> <li>• Ensure microphone is fully connected to MIC jack.</li> <li>• Try another microphone.</li> </ul>
No Blue Light when amplifier ON	<ul style="list-style-type: none"> <li>• Repress the Power(Push) knob.</li> <li>• Charge the amplifier.</li> </ul>
Distortion	<ul style="list-style-type: none"> <li>• Replace microphone.</li> <li>• Ensure microphone is fully connected to MIC jack.</li> </ul>
Feedback (squealing)	<ul style="list-style-type: none"> <li>• <u>Ensure microphone element is pointed away from amplifier speaker.</u></li> <li>• Reduce amplifier volume.</li> <li>• Ensure microphone is close to your upper lip.</li> <li>• Direct amplifier away from microphone.</li> </ul>
Installing neck strap	<ul style="list-style-type: none"> <li>• Use a toothpick to help get the strap through the sliver slot.</li> </ul>

### Warranty

The Griffin Laboratories Limited Warranty covers manufacturer defects in materials and workmanship. Coverage is for 1 year on the SoniVox® Plus™ amplifier, 90 days on included accessories.